


# MULTI-PURPOSE DRAINAGE SYSTEM

Contents of unopened, undamaged package are:  
**STERILE • NONPYROGENIC**

**DISPOSABLE** - Destroy after single patient use. Do not clean or resterilize.

**CAUTION:** Federal (U.S.A.) Law restricts this device to sale by or on the order of a physician. Dispose of all bodily fluids according to physician's instructions.

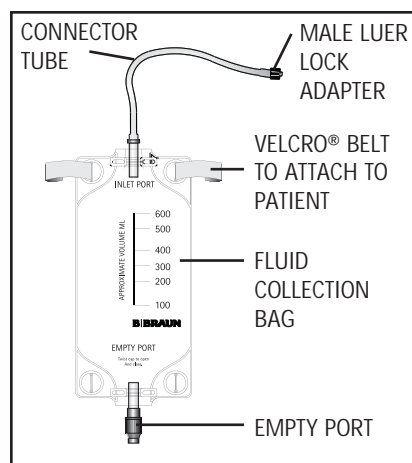

**B | BRAUN**

**B. Braun Medical Inc.**  
 Bethlehem, PA 18018

A4700594




P-3886

REV. 5/97

## DIRECTIONS:


Use Aseptic Technique.

1. Close the port at the bottom of the bag by twisting the adapter in a clockwise direction.
2. Firmly attach adapter on the drainage catheter to the connector tube. The length of the connector tubing can be altered by removing the adapter, trimming the connector tubing, and firmly reattaching the adapter.
3. Attach the bag to the patient below the drainage site. The bag may be worn either around the waist, or around the leg.
4. The bag can be opened as necessary for emptying. Twist the adapter at the bottom of the bag to open, then twist it to close after emptying.

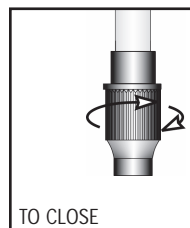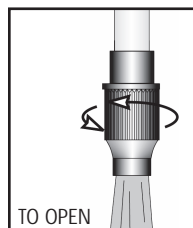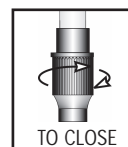


## PLACEMENT OF THE BAG:

1. The bag may be worn around the waist. Fasten the Velcro® belt around the waist of the patient.

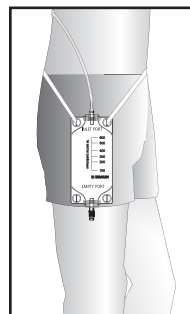


SEE OTHER SIDE OF INSTRUCTION SHEET

- The bag may be worn around the thigh. To prepare the Velcro®, cut the Velcro® band with a scissors. Be sure to leave enough length of Velcro® to secure around the upper thigh.

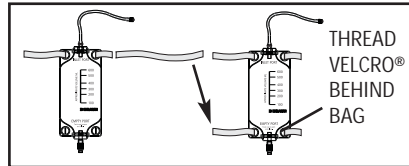

- Thread the piece of Velcro® you cut off through the eyelets at the bottom of the bag. Be sure to thread the Velcro® behind the empty port of the bag and not in front to allow free-flow of fluid from the bag.
- Secure the bag to the leg.

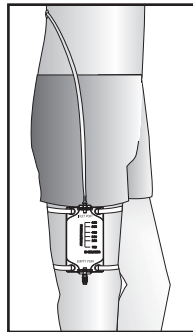

Velcro® is a registered trademark of Velcro USA Inc.