

PRECAUTIONS

- This is intended for use by physicians trained and experienced in diagnostic and interventional techniques. Standard techniques for placement of vascular access sheaths, angiographic catheters and guidewires should be employed.
- Manipulation of the product requires fluoroscopic control.
- Excessive forces should not be used to manipulate or retrieve foreign objects.
- Do not attempt to shape the catheter tip or basket, as doing so may damage the device. Damage may include, but not limited to, a compromise of the inner lumen diameter.
- The $\bar{O}\bar{N}\bar{O}$ is not indicated for pressure injection. The $\bar{O}\bar{N}\bar{O}$ allows for hand injections performed with a 10 mm or larger syringe.

POTENTIAL ADVERSE EVENTS

Potential adverse procedural events related to foreign body manipulation and retrieval in the vasculature can include, but are not limited to:

- Embolization
- Pulmonary embolism
- Vessel perforation
- Stroke
- Myocardial infarction
- Device entrapment

$\bar{O}\bar{N}\bar{O}$ Retrieval Device

DEVICE DESCRIPTION

The 12F $\bar{O}\bar{N}\bar{O}$ Retrieval Device ($\bar{O}\bar{N}\bar{O}$) consists of a series of braided loops forming a basket fused to a 7.5F inner diameter catheter. A peel away sheath is preloaded over the $\bar{O}\bar{N}\bar{O}$ shaft which will assist in compression of the basket for introduction through the hemostasis valve of a 12F or larger vascular sheath.

INDICATIONS FOR USE

The $\bar{O}\bar{N}\bar{O}$ Retrieval Device is indicated for use in the cardiovascular system to retrieve foreign objects using minimally invasive surgical procedures. Procedures include retrieval of intravascular foreign objects such as coils, balloons, catheters, guidewires, and/or filters within the cardiovascular system. This device is not intended for use in the coronary arteries or neurovasculature.

CONTRAINDICATIONS


This device is not intended for the removal of foreign objects entrapped by tissue growth.

WARNINGS

This product contains nitinol, a nickel-titanium alloy. Allergic reactions to nickel should be considered.

CAUTION: US federal law restricts this device to sale by or on the order of a physician (or properly licensed practitioner).



 Manufactured by:
ONOCOR LLC
Philadelphia, PA 19146
www.onocorvascular.com

US PATENTS 10,939,938; 10,092,324;
11,051,853

Interventional Systems
B | BRAUN

Distributed by:
B. Braun Interventional Systems, Inc.
Bethlehem, PA 18018-3524
Customer Service:
TEL: (877) 836-2228, FAX: (610) 849-1334
www.bisusa.com

L0001 Rev E April 2023

INSTRUCTIONS FOR USE

1. Remove the $\bar{O}\bar{N}\bar{O}$ device from packaging.
2. Advance the peel away sheath over the braided basket.
3. Flush the peel away sheath lumen with heparinized saline.
4. Flush the catheter lumen with heparinized saline.
5. Insert the tip of the peel away sheath into the retrieval sheath just past the hemostasis valve (minimum diameter 12F; maximum length 75 cm). Advance the basket into the lumen of the retrieval sheath.
6. If desired, remove the peel away sheath from the shaft of the $\bar{O}\bar{N}\bar{O}$.
7. Under fluoroscopic guidance advance the $\bar{O}\bar{N}\bar{O}$ through the retrieval sheath and deploy the basket adjacent to the foreign body to be retrieved. Note that the basket shortens upon deployment affecting the overall and working lengths of the system.

Basket Compressed OD	ONO Overall Length	ONO Working Length
10 mm	98 cm	94 cm
20 mm	96 cm	92 cm
Unconstrained	90 cm	86 cm

















8. Advance a snare or other manipulation device through the lumen of the $\bar{O}\bar{N}\bar{O}$ (maximum diameter 7F; minimum length 100 cm).
9. Grasp the targeted foreign body with the snare and retract it into the basket. **Note: if resistance is felt, re-assess the position of the foreign body and snare before proceeding.**
10. Stabilize the retrieval sheath and withdraw the snare and foreign body into the retrieval sheath simultaneously. **Note: if resistance is felt, re-assess the position of the foreign body and snare before proceeding.**
11. Remove the foreign object from the retrieval sheath or remove the foreign body / retrieval sheath simultaneously. **Note: if resistance is felt, re-assess the position of the foreign body and snare before proceeding.**
12. After retrieval, hospital standard of care should be followed for removing the sheath and providing hemostasis to prevent bleeding at the vascular access site. Dispose of all used devices in accordance with hospital policy for biohazardous materials.

HOW SUPPLIED

Supplied sterilized by ethylene oxide gas in peel-open packages. Intended for one-time use. Sterile if package is unopened and undamaged. Do not use the product if there is doubt as to whether the product is sterile. Store in a dark, dry, cool place. Avoid extended exposure to light. Upon removal from package, inspect the product to ensure no damage has occurred.

After use, dispose of product and packaging in accordance with hospital, administrative and/or local government policy.

IDENTIFICATION OF SYMBOLS USED ON PRODUCT LABELING

	Reorder Number		Sterilized using Ethylene Oxide		Non-pyrogenic
	Lot Number		Single Use		Prescription Device
	Date of Manufacture		Do not resterilize		Manufacturer
	Use by Date		Do not use if package is damaged and consult instructions for use		MR Unsafe
	Single sterile barrier system with protective packaging outside		Keep dry		Consult instructions for use
	Single sterile barrier system	Not made with natural rubber latex or DEHP			