

ACCEL™ Connection Set with Optional Non-Vented Spike

for the aspiration, removal, or
sampling of body fluids using an
external vacuum source




REF 470190
LENGTH 50.5 in. (128 cm) (approx)
PRIMING VOLUME 4.8 mL (approx)
GTIN 04046964716294


 **PRECAUTION:** Not intended for use with power injectors.

Instructions for Use: Use Aseptic Technique.
• Replace set per CDC guidelines.
1. Squeeze pinch clamp to closed position.
2. Firmly attach connector(s) to desired connection(s).
3. If required, attach non-vented spike adaptor to distal connector.
4. Open pinch clamp to start aspiration, removal, or sampling of body fluids.

STERILE EO Contents sterile in unopened, NONPYROGENIC undamaged package.

Not made with natural rubber latex or DEHP.

 For single use only. Do not resterilize.

 Do not use if package is damaged.


Keep away from sunlight.

Rx only
Pictorial is for reference only.


Keep dry.

P-8284-3 REV. 8/16



Manufactured for:
**B. Braun Interventional
Systems Inc.**
824 Twelfth Avenue
Bethlehem, PA 18018
www.bisusa.org

Customer Service, ordering
TEL: (877) VENA-CAV
(836-2228)
FAX: 1-(610)-849-1334
Assembled in Dominican Republic.
Components from USA and Italy.