

# ACCEL™ Connection Set with Closed Valve

for the aspiration, removal, or  
sampling of body fluids using  
an external vacuum source

**REF** 470191

**LENGTH** 50.5 in. (128 cm) (approx)

**PRIMING VOLUME** 5.8 mL (approx)

GTIN 04046964547980



**PRECAUTION:** Not intended  
for use with power injectors.

**Instructions for Use:** Use Aseptic  
Technique.

- Replace set per CDC guidelines.
- 1. Squeeze pinch clamp to closed position.
- 2. Firmly attach connector(s) to desired connection(s).
- 3. Open pinch clamp to start aspiration, removal, or sampling of body fluids.

**STERILE EO**  
NONPYROGENIC

Contents sterile in unopened,  
undamaged package.



Keep away  
from sunlight.



Keep dry.

Not made with natural  
rubber latex or DEHP.



Do not use if package  
is damaged.

Pictorial is for reference only.

P-8285-3 REV. 8/16

Rx only



For single use only.  
Do not resterilize.



**Manufactured for:**  
**B. Braun Interventional  
Systems Inc.**  
824 Twelfth Avenue  
Bethlehem, PA 18018  
www.bisusa.org

Customer Service, ordering  
TEL: (877) VENA-CAV  
(836-2228)  
FAX: 1-(610)-849-1334  
Assembled in Dominican  
Republic.