

ACCEL™ Split Connection Set with Closed Valve


for the aspiration, removal, or sampling of body fluids using an external vacuum source

REF 470192

LENGTH 47.5 in. (121 cm) (approx)

PRIMING VOLUME 6.1 mL (approx)

GTIN 04046964548000

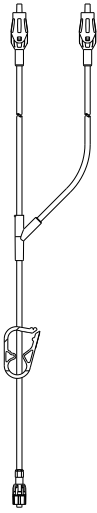

PRECAUTION: Not intended for use with power injectors.

Instructions for Use: Use Aseptic Technique.

- Replace set per CDC guidelines.
 1. Squeeze pinch clamp to closed position.
 2. Firmly attach connector(s) to desired connection(s).
 3. Open pinch clamp to start aspiration, removal, or sampling of body fluids.

STERILE EO
NONPYROGENIC

Contents sterile in unopened, undamaged package.

Not made with natural rubber latex or DEHP.



Do not use if package is damaged.



Keep away from sunlight.



Keep dry.

Rx only



For single use only. Do not resterilize.

Pictorial is for reference only.

P-8286-3
REV. 8/16



Manufactured for:
B. Braun Interventional Systems Inc.
824 Twelfth Avenue
Bethlehem, PA 18018
www.bisusa.org

Customer Service, ordering
TEL: (877) VENA-CAV
(836-2228)
FAX: 1-(610)-849-1334
Assembled in Dominican Republic.