

Application

Using the ASEPT® Evacuated Drainage Bottle with Drain Line

Follow these step-by-step directions.

1. Pick up the bag that contains the ASEPT® 1,000ml or 600ml Evacuated Drainage Bottle with Drain Line. Look at the green vacuum indicator on the top of the bottle. This indicator tells you if there is a vacuum in the bottle. If the indicator is up and level with the "min" line then discard the bottle and use another.
2. Open the peel package under sterile conditions and take out the ASEPT® 1,000ml or 600ml Evacuated Drainage Bottle with Drain Line.
3. Remove the twist tie that holds the drain line together.
4. Squeeze the pinch clamp on the drain line completely closed.
5. Remove the protective cap from the luer connector on the drain line and align it in the center of the opposing luer and twist together to ensure the connection is closed.
6. Slide the white clamp on the green tubing to release the vacuum.
7. Release the pinch clamp on the drain line, slowly, to begin draining the fluid. The fluid will then flow into the Evacuated Drainage Bottle.
8. After the drainage procedure; squeeze the pinch clamp on the drain line fully closed and disconnect the drain line connection. The needle-free sampling port may be used for fluid sampling during or after drainage procedures. To sample fluid, swab the needle-free sampling port with a disinfectant and use a needle-free syringe to access the needle-free sampling port, hold the bottle upside down and remove the fluid.
9. Dispose the ASEPT® 1,000ml or 600ml Evacuated Drainage Bottle with Drain Line in accordance with the facility's policies and procedures.

STERILITY

This device has been sterilized, and is for single use only and is not to be reused. As long as the packaging remains sealed and uncompromised the contents within each package are sterile. pfm medical will not be responsible for any products that are resterilized, nor accept for exchange or credit any product that has been opened but not used by the patient or purchaser.

Warranty

pfm medical warrants that this medical device is free from defects in both materials and workmanship. The above warranties are in lieu of all other warranties, either expressed or implied, including any warranty of merchantability or fitness for a particular purpose. Suitability for use of the medical device for any surgical procedure shall be determined by the user. pfm medical shall not be liable for incidental or consequential damages of any kind.



pfm medical, inc.
1916 Palomar Oaks Way, Suite 150
Carlsbad, CA 92008
Tel.: (760) 758-8749
Fax: (760) 758-1167
www.pfmmedicalusa.com
customerservice@pfmmedicalusa.com

LS-00200-01-AC 2019-04

pfm medical
Quality and Experience

ASEPT®

**1,000ml & 600ml Evacuated
Drainage Bottle with Drain Line
Instructions for Use**

www.pfmmedicalusa.com

Explanation of the symbols on label and packaging



Catalog number



Batch code



Consult Instructions for Use



Sterilized using Ethylene Oxide



Keep away from sunlight



Keep dry



Do not Resterilize



Use-by date



Do not re-use

Rx only

Prescription only



Do not use if package is damaged



Manufacturer

Instructions for Use

ASEPT[®] 1,000ml & 600ml Evacuated Drainage Bottle with Drain Line (Order# M7050, M7052)

Product Description

The ASEPT[®] 1,000ml or 600ml Evacuated Drainage Bottle with Drain Line is a vacuum based suction device for drainage purposes. The bottle comes with a drain line, a needle-free sampling port, and a luer connector at the end of the drain line. The following instructions are for the ASEPT[®] 1,000ml or 600ml Evacuated Drainage Bottle with Drain Line.

The ASEPT[®] 1,000ml & 600ml Evacuated Drainage Bottle with Drain Line contains the following items:

One 1,000ml or 600ml vacuum bottle with the following features:

- Bottle scaled in 10ml increments
- Green vacuum indicator
- Clear drain tubing with pinch clamp
- Needle-free sampling port
- Vacuum release clamp
- Luer connector with protective cap

General Information

Cautions

- ⚠ The green vacuum indicator should be down and level with the “max” line; this indicates that the inside of the bottle contains a vacuum
- ⚠ Before use, make sure the protective cap is on the luer connector at the end of the drain line
- ⚠ When the green vacuum indicator reaches the position “min” a change of bottle is necessary
- ⚠ Federal Law (USA) restricts this device to sale by or on the order of a physician
- ⚠ This product contains dry natural rubber

Indications for Use

The ASEPT[®] Evacuated Drainage Bottle with Drain Line is a non-powered, single patient, portable suction device used to aspirate, remove or sample body fluids.

Indications for Use - ASEPT[®] Peritoneal Drainage System

The ASEPT[®] Peritoneal Drainage System is indicated for periodic drainage of recurrent and symptomatic malignant ascites. The catheter is intended for long term access of the peritoneal cavity in order to relieve symptoms such as dyspnea.

Indications for Use - ASEPT[®] Pleural Drainage System

The ASEPT[®] Pleural Drainage System is intended for long term, intermittent drainage of symptomatic, recurrent, pleural effusions, including malignant pleural effusions that do not respond to treatment of the underlying disease.